




## Low Profile Anesthesia Facemasks

### Package Contents

- 5 - facemasks (small or medium)
- 1 - tubing set with facemask fitting (1-meter)

#### Tubing Adapters:

- 2 each - (1/4" and 3/8" tube adapters with traditional kits)
- 2 each – male luer fittings with SomnoSuite kits

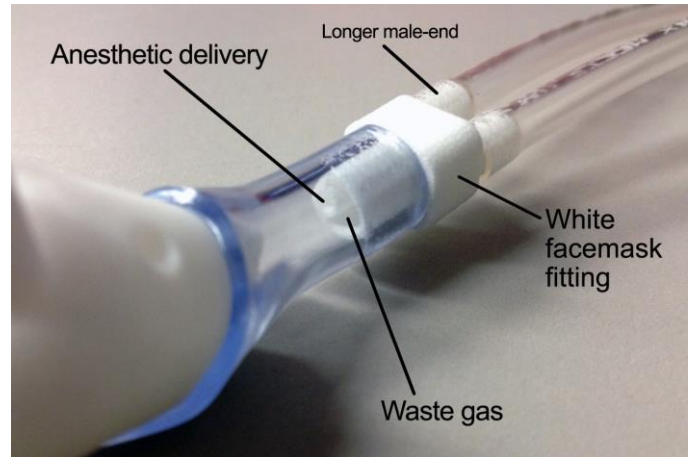

### Instructions for changing the facemask

NOTE: Each kit includes one facemask already attached to the fitting, which is attached to the tubing.

1. Remove the existing facemask from the white facemask fitting.
2. Select a facemask from the package.
3. Firmly press the facemask onto the white facemask fitting.
4. Connect the facemask tubing to the vaporizer, if necessary use the included adapters.

NOTE: The longer male-end on the back of the facemask fitting identifies the anesthetic delivery tubing which should always be connected to the vaporizer. The shorter male-end should be connected to your waste gas management system.

### Tips for a Good Fit

- The facemask can be used in any orientation; simply rotate the mask on the white facemask fitting
- Ensure the animal's nose and mouth are covered
- The facemask should fit snugly around the animal's muzzle without making contact with the animal's eyes
- Each animal has a unique muzzle, if a good fit is difficult to obtain try using a different mask from the same package